Inspected Dimensions:  
Folded width: 5-1/2" (14 cm)  
Folded length: 7-1/2" (19 cm)

**ARROW®**

## Rapid Infusion Catheter (RIC®) Exchange Product

### Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged. **Warning: Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.**

Do not alter sheath or any kit/set components during insertion, use or removal.

Procedure must be performed by trained personnel well versed in anatomical landmarks, safe technique and potential complications.

### Indications for Use:

The Arrow Rapid Infusion Catheter (RIC®) permits exchange of a 20 Ga. (or larger) peripheral intravenous catheter to a large diameter sheath for rapid volume infusion.

### Contraindications:

None known.

### Warnings and Precautions:

- Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.**
- Warning: Practitioners must be aware of complications associated with peripheral large bore catheters including vessel wall perforation, infiltration, catheter embolism, infection, thrombosis, inadvertent arterial puncture, nerve damage, hematoma, intravascular clotting, and hemorrhage.**
- Warning: Use appropriate size sheath for intended exchange site. Vessel rupture is possible if the diameter of the sheath is too large in comparison to the vessel diameter at the peripheral exchange site.**
- Warning: To lessen the risk of disconnects, only securely tightened Luer-Lock connections should be used with this device.**
- Warning: Do not apply excessive force in removing guide wire, dilator or sheath. If**

**withdrawal cannot be easily accomplished, an x-ray should be obtained and further consultation requested.**

- Warning: Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.**
- Precaution: Do not suture directly to the outside diameter of the sheath to minimize the risk of cutting or damaging the sheath or impeding sheath flow.**
- Precaution: Indwelling sheaths should be routinely inspected for desired flow rate, security of dressing, correct position and for proper Luer-Lock connection.**

### A Suggested Procedure: Use sterile technique.

- Cleanse puncture site, indwelling peripheral catheter and intravenous administration tubing connector with appropriate antiseptic agent.
- Disconnect tubing from catheter and insert spring-wire guide the required distance into catheter. **Precaution: Maintain firm grip on guide wire at all times. Precaution: If resistance is encountered while advancing guide wire, hold catheter in place and carefully withdraw guide wire. Attempt to reinsert. If resistance is encountered again, abort procedure.**
- With spring-wire guide in place within the vein, remove indwelling peripheral catheter over spring-wire guide.
- Enlarge cutaneous puncture site with cutting edge of scalpel positioned away from spring-wire guide. **Precaution: Do not cut guide wire.**
- Remove plastic guard from sheath/dilator assembly.
- Thread tapered tip of dilator over spring-wire guide. Grasping near skin, advance dilator and sheath into vessel with a slight twisting motion.
- Advance sheath over dilator into vessel, again grasping near skin and using slight twisting motion.

8. Holding sheath in place, remove guide wire and dilator as a unit. **Precaution: Do not leave vessel dilator in place as an indwelling catheter to minimize the risk of possible vessel wall perforation.**
9. Attach desired Luer-Lock administration tubing to sheath hub. For best results large diameter fluid administration tubing should be used. **Precaution: To minimize the risk of disconnects use only Luer-Lock connecting tubing.**
10. Secure sheath to patient using suture wing and/or anchor with a purse string suture around the sheath suture ring. **Precaution: Do not suture directly to the outside diameter of the sheath to minimize the risk of cutting or damaging the sheath or impeding sheath flow.**
11. Dress puncture site per hospital protocol.
12. Record the exchange procedure on the patient's chart.

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