STERILE R

tyco / Healthcare

KENDALL

MAGELLAN™

Syringe with Safety Needle

Fig. 1A

FINGER ACTIVATION

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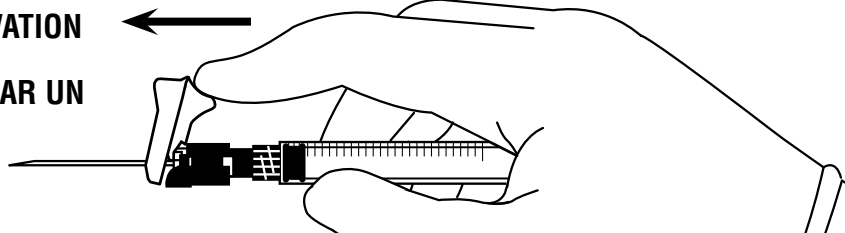
Fig. 1B

THUMB ACTIVATION

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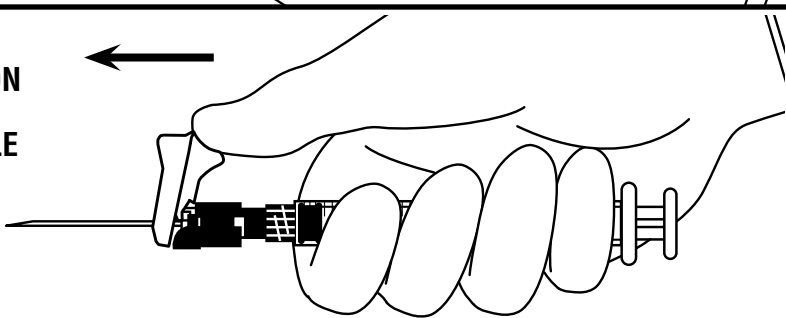
Fig. 2

SURFACE ACTIVATION

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
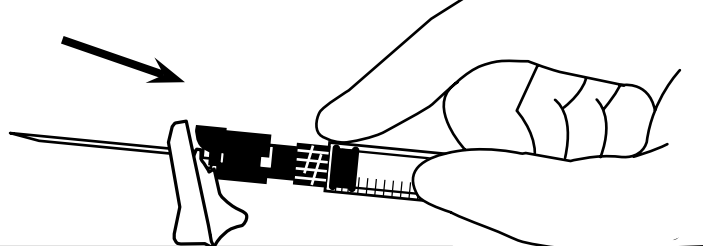
Fig. 3

NEEDLE CONNECT / DISCONNECT

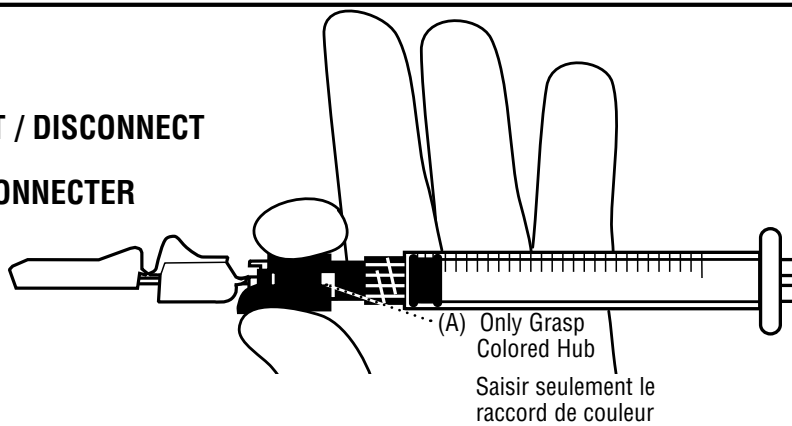CONNECTER/DÉCONNECTER  
L'AIGUILLE

MAGELLAN MAXDOSE needle is optimized for use with the MONOJECT 1mL luer lock syringe and as such is not compatible with any other syringe.

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Covered by one or more of the following patents:

U.S. Patent Nos. 5,823,997 and 5,980,488.

Other U.S., foreign and regional patents granted and pending.

™Trademark of TycoHealthcare Group LP or its affiliate

TYCO HEALTHCARE GROUP LP • MANSFIELD, MA 02048

MADE IN U.S.A. • © 2007 TYCO HEALTHCARE GROUP LP

ALL RIGHTS RESERVED • 070613 • PRODUCT INFORMATION 1-800-962-9888

www.tycohealthcare.com

Rx  
ONLY

Single Use

English

MAGELLAN Safety Needle with Syringe

Sterility of unit package is guaranteed unless package is opened or damaged.

Do not use if unit package or product is damaged or open.

Each MAGELLAN Syringe with Safety Needle is latex-free, non-toxic, non-pyrogenic, and for single use only.

INSTRUCTIONS FOR USE:

1. To open package, peel back paper tabs completely exposing the MAGELLAN Safety Needle and Syringe combination. Use aseptic technique.
2. Twist syringe clockwise while holding the rigid sheath that covers the MAGELLAN Safety Needle to seat the Safety Needle to the syringe.
3. Remove the sheath from the MAGELLAN Safety Needle while directing the needle away from you and others. Be certain not to touch the needle or the needle safety shield.
4. If performing injection, draw medication into syringe and use aseptic technique.
5. If necessary to transport filled syringe to point of administration, use a safe, passive recapping technique to cover the needle before transport. In accordance with OSHA standards, such recapping must be accomplished by a one-handed technique, i.e., do not hold the needle shield during the recapping process.
6. Perform injection (or aspiration of body fluid) according to local standard procedure and aseptic technique.
7. Immediately following the injection or aspiration procedure, lock the safety needle shield using any of the following methods:
  - Push the safety shield forward in a single-handed manner to cover the needle and lock the shield. Keep your finger (Fig. 1A — FINGER ACTIVATION) or thumb (Fig. 1B — THUMB ACTIVATION) behind the needle tip at all times.
- OR
- Press the safety shield, lever side down, onto a flat surface to cover the needle and lock the shield, i.e., bedside table (Fig. 2 — SURFACE ACTIVATION).
8. The MAGELLAN Safety Needle is locked once the needle tip is completely shielded. Verify locked position through audible, tactile, and/or visual feedback.
9. **ONLY GRASP THE COLORED HUB TO CONNECT/DISCONNECT NEEDLE FROM SYRINGE** (see Fig. 3 — LOCKED NEEDLE SHIELD). (A) = Colored Hub. (Only Grasp Colored Hub.)
10. Once safety shield is locked, immediately discard the MAGELLAN Safety Needle and syringe into the nearest sharps container following applicable regulations.

**CAUTION: DO NOT AUTOCLAVE BEFORE USE.**

**Keep hands away from needle tip at all times.**

**Do not attempt to override or defeat the safety locking mechanism.**

**To help avoid infectious diseases due to accidental needlesticks, contaminated needles should not be recapped or removed, unless there is no alternative or such action is required by a medical procedure.**

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