

# ARROW®

## SharpsAway II™ Locking Disposal Cup

### Indications for Use:

The Arrow® SharpsAway II™ Locking Disposal Cup is intended as a single use needle protection device, which covers the end of a needle after use to help prevent needle sticks.

### Contraindications:

None known.

### Warnings and Precautions:\*

- 1. Warning:** Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.
- 2. Precaution:** Properly dispose of sharps in sharps container in accordance with state/federal OSHA standards for blood borne pathogens and/or institutional policy.

### A Suggested Procedure: Use sterile technique.

- The SharpsAway II™ Locking Disposal Cup is used for disposal of needles (15 Ga. - 30 Ga.).
  - Using one-handed technique, firmly push needles into disposal cup holes (refer to Fig. 1).

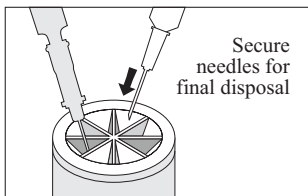


Fig. 1

- Once placed into disposal cup, needles will be automatically secured in place so that they cannot be reused.
  - Discard entire cup, at completion of procedure, into an approved sharps container.
- Precaution:** Do not attempt to remove needles that have been placed into cup. These needles are secured in place. Damage may occur to needles if they are forced out of disposal cup.

# ARROW®

INTERNATIONAL  
2400 Bernville Road  
Reading, PA 19605 USA  
1-800-523-8446  
1-610-378-0131  
8 a.m. - 8 p.m. EST