Inspected Dimensions:
Folded width: 5-1/2" (14 cm)
Folded length: 7-1/2" (19 cm)

ARROW®

Arterial Catheterization Product

Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged.

Warning: Read all package insert warnings, precautions, and instructions prior to use. Failure to do so may result in severe patient injury or death.

Do not alter catheter or any other kit/set component during insertion, use or removal.

Procedure must be performed by trained personnel well versed in anatomical landmarks, safe technique and potential complications.

Indications for Use:

The Arrow Arterial Catheterization Device permits access to peripheral arterial circulation.

Contraindications:

Inadequate collateral arterial circulation.

Warnings and Precautions:*

1. **Warning:** Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.
2. **Warning:** Practitioners must be aware of complications associated with arterial procedures^{1,7,8,11} including bacteremia,² septicemia, vessel wall perforation, intravascular clotting and embolization, hematoma, arterial spasm, tissue necrosis, hemorrhage, thrombosis, peripheral ischemia and infarction, peripheral nerve damage⁹ and air embolism.³
3. **Warning:** In brachial procedures collateral flow cannot be guaranteed, and therefore intravascular clotting can result in tissue necrosis.¹²
4. **Warning:** In radial artery procedures, practitioners must ascertain that definite evidence of collateral ulnar flow exists.
5. **Warning:** Care should be exercised that the indwelling catheter is not inadvertently kinked at the hub area when securing catheter to the patient. Kinking may weaken the wall of the catheter and cause a fraying or fatigue of the material, leading to possible separation of the catheter.
6. **Warning:** Do not suture directly to outside diameter of catheter body to minimize the risk of damaging the catheter or adversely affecting monitoring capabilities.
7. **Warning:** Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.
8. **Precaution:** In order to minimize the risk of problems associated with disconnects, it is recommended that only Luer-Lock connecting tubing be used with this device.
9. **Precaution:** Alcohol and acetone can weaken the structure of polyurethane materials. Check ingredients of prep sprays and swabs for acetone and alcohol content.
Acetone: Do not use acetone on catheter surface. Acetone may be applied to skin but must be allowed to dry completely prior to applying dressing.

Alcohol: Do not use alcohol to soak catheter surface or to restore catheter patency. Care should be taken when instilling drugs containing high concentration of alcohol. Always allow alcohol to dry completely prior to applying dressing.

10. **Precaution:** Indwelling catheter should be routinely inspected for desired flow rate, security of dressing, and possible migration. Do not use scissors to remove dressing to minimize the risk of cutting catheter.
11. **Precaution:** Remove insert tube prior to use where applicable.

A Suggested Procedure:

Use sterile technique.

1. Assess for adequate collateral circulation.
2. Prep and drape anticipated puncture site per hospital protocol.
3. Utilize local anesthetic as required.
4. **Precaution:** Remove insert tube from catheter where applicable. Place on sterile field.
5. Locate and puncture artery with desired introducer needle or introducer catheter/needle assembly, where provided. Avoid transfixing both vessel walls. **Precaution:** If both vessel walls are punctured, subsequent advancement of spring-wire guide could result in inadvertent sub-arterial placement. Pulsatile blood flow from the needle hub indicates successful entry into artery.
6. Insert desired tip of spring-wire guide through introducer needle into artery. If "J" tip spring-wire guide is used, prepare for insertion by sliding plastic tube over the "J" to straighten. Advance spring-wire guide to required depth. Advancement of "J" tip may require a gentle rotating motion. **Precaution:** Maintain firm grip on spring-wire guide at all times.
7. Hold spring-wire guide in place and remove introducer needle. **Warning:** Do not withdraw spring-wire guide against needle bevel to avoid possible severing of spring-wire guide.
8. Enlarge puncture site with scalpel if desired. **Warning:** Do not cut spring-wire guide.
9. Thread tip of indwelling catheter over spring-wire guide. **Precaution:** Allow sufficient guide wire length to remain exposed at hub end of catheter to maintain firm grip on guide wire. Grasping near skin, advance catheter into vessel with slight twisting motion.
10. Hold catheter at desired depth and remove spring-wire guide. Pulsatile blood flow indicates positive arterial placement. **Precaution:** Use care when removing spring-wire guide. If resistance is encountered, remove spring-wire guide and catheter simultaneously. Use of excessive force may damage catheter or spring-wire guide.
11. Attach desired Luer-Lock stopcock, injection site or connecting tubing to catheter hub. Secure catheter to patient in preferred manner using suture wings, suture groove or wing clip, where provided. **Warning:** Care should be exercised that the indwelling catheter is not inadvertently kinked at the hub area when securing catheter to the patient. Kinking may weaken the wall of the catheter and cause a fraying or fatigue of the material, leading to possible

separation of the catheter. **Warning: Do not suture directly to outside diameter of catheter body to minimize the risk of damaging the catheter or adversely affecting monitoring capabilities.**

12. Dress puncture site according to hospital protocol.

Catheter Removal Procedure:

1. Remove dressing. **Precaution: To avoid cutting of the catheter do not use scissors to remove the dressing.**
2. **Warning: Exposure of the arterial circulation to atmospheric pressure may result in entry of air into the circulation.** Remove suture(s). Be careful not to cut the catheter. Remove catheter slowly, pulling it parallel to the skin. As catheter exits the site, apply pressure with a dressing until hemostasis occurs. Apply light pressure dressing to site.
3. Upon removal of the catheter, inspect it to make sure that the entire length has been withdrawn.
4. Verify that the catheter was intact upon removal. Document removal procedure.

References:

1. Abadir AR, Ung KA. Complications of radial artery cannulation. *Anesthesiology Rev*: 1980;7:11-16.
2. Band JD, Maki DG. Infections caused by arterial catheters used for hemodynamic monitoring. *Am J Med*. 1979;67:735-741.
3. Chang C, Dughi J, Shitabata P, Johnson G, Coel M, McNamara JJ. Air embolism and the radial arterial line. *Crit Care Med*. 1988;16:141-143.
4. Clark CA, Harman EM. Hemodynamic monitoring: arterial catheters. In: Taylor RW, Civetta JM, Dirby RR, eds.

Techniques and Procedures in Critical Care. Philadelphia, PA: JB Lippincott; 1990:218-230.

5. Daily EK, Schroeder JS. *Techniques in Bedside Hemodynamic Monitoring*. 5th ed. St. Louis, MO: Mosby; 1994:71-77.
6. Falk PS, Scuder PE, Sherertz RJ, Motsinger SM. Infected radial artery pseudoaneurysms occurring after percutaneous cannulation. *Chest*. 1992;101:490-495.
7. Kaye W. Invasive monitoring techniques: arterial cannulation, bedside pulmonary artery catheterization, and arterial puncture. *Heart Lung*. 1983;12:395-427.
8. Kaye WE, Dubin HG. Vascular cannulation. In: Taylor RW, Civetta JM, Dirby RR, eds. *Techniques and Procedures in Critical Care*. Philadelphia, PA: JB Lippincott; 1990:204-208.
9. Marshall G, Edelstein G, Hirshman CA. Median nerve compression following radial arterial puncture. *Anesth Analg*. 1980;59:953-954.
10. Norwood SH, Cormier B, McMahon NG, Moss A, Moore V. Prospective study of catheter related infection during prolonged arterial catheterization. *Crit Care Med*. 1988;16:836-839.
11. Puri VK, Carlson RW, Bander JJ, Weil MH. Complications of vascular catheterization in the critically ill. *Crit Care Med*. 1980;8:495-499.
12. Venus B, Satish P. Vascular Cannulation. In: Civetta JM, Taylor RW, Kirby RR, eds. *Critical Care*. 3rd ed. Philadelphia, PA: Lippincott-Raven Publishers; 1997:539-542.

Arrow International, Inc. recommends that the user be acquainted with the reference literature.

*If you have any questions or would like additional reference information, please contact Arrow International, Inc.

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Arrow International, Inc. rekommenderar att användaren studerar referenslitteraturen.
*Om du har frågor eller önskar ytterligare referensinformation, kontakta Arrow International, Inc.