Inspected Dimensions:
Folded width: 5-1/2" (14 cm)
Folded length: 7-1/2" (19 cm)

ARROW®

Twin-Cath® Two-Lumen Intravenous Catheter

Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged. **Warning: Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.**

Do not alter the catheter or any other kit/set component during insertion, use, or removal.

Procedure must be performed by trained personnel well versed in anatomical landmarks, safe technique and potential complications.

Indications for Use:

The Arrow Twin-Cath® permits peripheral venous access.

Contraindications:

None known.

Warnings and Precautions:*

1. **Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.**
2. **Warning: Practitioners must be aware of complications associated with peripheral intravenous catheters^{4,5} including infiltration, catheter embolus, bacteremia, septicemia, thrombosis, inadvertent arterial puncture, nerve damage, hematoma formation, and hemorrhage.**
3. **Warning: To lessen the risk of disconnects, only securely tightened Luer-Lock connections should be used with this device.**
4. **Warning: Do not use scissors to remove dressing to minimize the risk of damage to the catheter.**
5. **Warning: Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.**
6. **Precaution: Limit placement of catheter to large venous channels of upper arm and forearm.**
7. **Precaution: Do not suture directly to the outside diameter of the catheter to minimize the risk of**

cutting or damaging catheter or impeding catheter flow.

8. **Precaution: Catheter should be routinely inspected for desired flow rate, security of dressing, correct position, and proper Luer-Lock connection.**
9. **Precaution: Maintain insertion site with regular meticulous redressing using aseptic technique.**
10. **Precaution: For blood sampling temporarily shut off distal port if solution is being infused.**
11. **Precaution: Alcohol and acetone can weaken the structure of polyurethane material. Therefore, care should be taken when instilling drugs containing alcohol or when using high concentration of alcohol or acetone when performing routine catheter care and maintenance. Alcohol should not be utilized to decontaminate polyurethane catheters.**

A Suggested Procedure:

Use sterile technique.

1. Prep puncture site as required.
2. Prepare catheter for insertion. Flush proximal port through injection cap. Leave guard in place, hold catheter upright, and flush. This will activate the catheter's hydrophilic coating. **Precaution: Do not allow flush solution to go beyond tip of catheter to reduce interference with blood flashback.**
3. Puncture vessel using a continuous, controlled, slow forward motion. Avoid transfixing both vessel walls. Blood flashback in hub of introducer needle indicates successful entry into vessel. Aspiration may be required.
4. Advance catheter and needle, as a unit, approximately one centimeter, to ensure that vessel dilation is complete.
5. Hold needle in position and advance catheter forward into vessel. Remove introducer needle. **Precaution: Do not reinsert needle into catheter to minimize the risk of catheter damage.**
6. Attach stopcock, injection cap, or connecting tubing to distal hub. **Precaution: To lessen the risk of disconnects, only securely tightened**

Luer-Lock connections should be used with this device. Precaution: Do not begin infusion until proximal lumen placement is verified.

7. Verify proximal lumen placement by aspirating blood from the proximal port through the extension line and flush.
8. Attach proximal hub to connecting line or heparin-lock through injection cap using standard hospital protocol. A clamp is provided to occlude flow through the proximal lumen. **Precaution: Open clamp prior to infusion.**
9. Secure catheter to patient. Suture wing or suture ring on juncture hub of catheter may be used. **Precaution: If using suture ring, suture should be snug but not tight. Tightening will constrict lumens.**
10. Dress puncture site per hospital protocol. **Precaution: Catheter should be routinely inspected for desired flow rate, security of dressing, correct position, and proper Luer-Lock connection.**

Infusate Guidelines^{1,2,3}:

1. Two diluted medications, continuous or intermittent, can be administered simultaneously.
2. One undiluted medication and one diluted medication can be administered simultaneously.
3. **Warning: Simultaneous administration of two undiluted medications is not recommended since there is clear *in-vitro* evidence of medication interaction at normal peripheral venous flow rates.**

References:

1. Chapolini RJ. Incompatible drug infusion through the Arrow two-lumen catheter: an *in-vitro* analysis. White Marsh Laboratories, Baltimore, MD; April 1987. Unpublished, data on file at Arrow International, Inc.
2. Chapolini RJ. Medication incompatibility and the Arrow Twin-Cath®: a follow-up. White Marsh Laboratories, Baltimore, MD; November 1988. Unpublished, data on file at Arrow International, Inc.
3. Jaimovich DG, Rose WW. *In vivo* evaluation of simultaneous administration of incompatible drugs via a double-lumen peripheral catheter. *Crit Care Med.* 1990;18:1164-1166.
4. Perdue M. Intravenous complications. In: Terry J, Baranowski L, Lonsway RA, Hedrick C., eds. *Intravenous Therapy*. Philadelphia, PA; WB Saunders; 1995:419-436.
5. Weinstein SM. *Plumer's Principles & Practice of Intravenous Therapy*. 5th ed. Philadelphia, PA; JB Lippincott Co; 1993:83-91.

Arrow International, Inc. recommends that the user be acquainted with the reference literature.

* If you have any questions or would like additional reference information, please contact Arrow International, Inc.

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Arrow International, Inc. rekommenderar att användaren studerar referenslitteraturen.
* Om du har frågor eller önskar ytterligare referensinformation, kontakta Arrow International, Inc.