



REF **DLX-45563-CURVB** (IPNXXXXXX)

6 Fr. **3** Lumen **55** cm catheter length **.018** inch dia. spring-wire guide

Pressure Injectable Arrowgard Blue Advance® Three-Lumen PICC pre-loaded with Arrow® NaviCurve™ Stylet



Contents:

- 1: Three Lumen TaperFree® Catheter with Arrowgard Blue Advance® Antimicrobial/Antithrombotic Protection¹: 6 Fr. (2.11 mm OD) x 55 cm, Pressure Injectable, Arrow® NaviCurve™ Stylet and T-Port Connector
- 1: GlideThru™ Peel-Away Sheath: 6 Fr. x 2-3/4" (7 cm) Radiopaque over 6 Fr. Dilator
- 1: Spring-Wire Guide, Nitinol, Marked: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on One End - Straight Stiff Tip on Other) with Arrow Advancer
- 1: Injection Needle: SafetyGlide™² 25 Ga. x 5/8" (1.60 cm) RW
- 1: Safety Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- 1: Blunt Fill Needle: 18 Ga. x 1-1/2" (3.81 cm)
- 1: Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- 1: Syringe: 5 mL Luer-Lock
- 4: 10 mL Luer-Lock Pre-Filled Saline Syringe
- 1: 5 mL 1% Lidocaine HCl Solution and Alcohol Prep
- 1: 3 mL Applicator 2% CHG and 70% IPA Chloraprep®³ One-Step Solution
- 2: 3 mL Applicator 2% CHG and 70% IPA Chloraprep®³ One-Step Solution with Hi-Lite Orange™ Tint
- 3: Dust Cap: Non-Vented
- 1: SharpsAway® Disposal Cup
- 1: SharpsAway® II Locking Disposal Cup
- 1: Catheter Trimmer
- 1: Towel: 24" x 36" (61 cm x 91 cm)
- 1: Maximal Barrier Drape™ with two 13 cm (5") x 15 cm (6") fenestrations
- 1: Safety Scalpel: #11
- 1: Sterile Procedure Sign
- 1: Patient ID Card
- 1: Patient Information Booklet
- 1: Medication Label: 1% Lidocaine
- 2: Paper Tape Measure
- 1: Tourniquet
- 1: Foam Electrodes (3 per pack)
- 10: Gauze Pad: 2" x 2" (5 cm x 5 cm)
- 10: Gauze Pad: 4" x 4" (10 cm x 10 cm)
- 1: Surgical Apparel: Impervious Gown
- 1: Dressing: I.V. Advanced 8.5 cm x 11.5 cm
- 1: Dressing: BIOPATCH®⁴
- 1: Transducer Cover with Gel and Rubber Bands
- 1: Remote Cover
- 1: Surgical Apparel: Mask with Eye Shield
- 1: Dressing: STATLOCK®⁵ PICC Plus Catheter Stabilization Device
- 1: Surgical Apparel: Bouffant Cap
- 1: Surgical Apparel: Mask
- 1: Tape: Steri-Strip®⁶
- 1: T-Piece Cover

¹Licensed under US Patent No. 7,329,412 and 11,219,706.
²A trademark of Becton, Dickinson and Company.
³A registered trademark of CareFusion or one of its subsidiaries.
⁴A registered trademark of Johnson & Johnson Corporation.
⁵A registered trademark of C. R. Bard, Inc.
⁶A registered trademark of 3M Company.

MR Safe
 Not made with natural rubber latex.

Store between 20 – 25°C (68 – 77°F).
 Arrow International provides the enclosed medication label(s) for your convenience. Please ensure that these labels are applied to the correct syringe and corresponding medication.

- Contraindications: The Pressure Injectable Arrowgard Blue Advance antimicrobial/antithrombotic catheter is contraindicated:**
- for patients with known hypersensitivity to chlorhexidine
 - in the presence of device related infection in the intended insertion vessel or catheter pathway
 - in the presence of thrombosis in the intended insertion vessel or catheter pathway



Lumen

	Priming Volume* (mL)	Gravity Flow Rate† (mL/hr)	Pump Flow Rate‡ (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Pink (18 Ga.)	0.70	990	5440	6
Blue (19 Ga.)	0.52	240	1930	not rated
White (19 Ga.)	0.49	270	2070	not rated

* Priming volumes are approximate and are done without accessories.
 † Flow rate values are approximate and are determined using deionized water at 100 cm head height.
 ‡ Pump flow rates are determined at maximum pump pressure of 10 psig and represent approximate flow capabilities.
 ** Pressure injection flow rates are determined at the injector pressure setting of 300 psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing.

