



REF PR-41552-BAS

5.5 Fr. 2 Lumen 15 cm catheter length .018 inch dia. spring-wire guide



# Pressure Injectable Arrowg+ard Blue Advance® Two-Lumen Midline

Contents:

- 1: Two-Lumen TaperFree® Catheter+++ with Arrowg+ard Blue Advance® Antimicrobial/Antithrombogenic Protection<sup>1</sup>: 5.5 Fr. (1.88 mm OD) x 15 cm Pressure Injectable with Blue FlexTip® Feature and Contamination Guard
- 1: GlideThru™ Peel-Away Sheath: 5.5 Fr. x 2-3/4" (7 cm) Radiopaque over 5.5 Fr. Dilator
- 1: Spring-Wire Guide, Nitinol, Marked: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on One End - Straight Stiff Tip on Other)
- 1: Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- 1: Syringe: 10 mL Luer-Lock
- 2: Dust Cap: Non-Vented
- 1: SecondSite™ Adjustable Hub: Fastener
- 1: SecondSite™ Adjustable Hub: Catheter Clamp
- 1: SharpsAway® Disposal Cup
- 1: SharpsAway® II Locking Disposal Cup
- 1: Catheter Trimmer
- 1: Safety Scalpel: #11
- 1: Overbed Sign
- 1: Patient Information Booklet
- 1: Paper Tape Measure
- 1: Dressing: STATLOCK®<sup>2</sup> PICC Plus Catheter Stabilization Device

<sup>1</sup>Licensed under US Patent No. 7,329,412 and 11,219,706.


<sup>2</sup>A registered trademark of C. R. Bard, Inc.

en fr-CA pt-BR es-MX

###MR Safe  
Not made with natural rubber latex.

Store below 25°C (77°F). Avoid excessive heat above 30°C (86°F).

**Contraindications:** The Arrowg+ard Blue Advance Pressure Injectable Midline is contraindicated for patients with known hypersensitivity to chlorhexidine.

 Lumen	Priming Volume* (mL)	Gravity Flow Rate† (mL/hr)	Pump Flow Rate†† (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Distal (18 Ga.)	0.30	1071	5860	5
Proximal (18 Ga.)	0.31	898	6660	not rated

\* Priming volumes are approximate and are done without accessories.  
† Flow rate values are approximate and are determined using deionized water at 100 cm head height.  
†† Pump flow rates are determined at maximum pump pressure of 10 psig and represent approximate flow capabilities.  
\*\* Pressure injection flow rates are determined using media of 11.8 centipoise (cP) viscosity, with 152 cm administration set tubing.

