



REF PR-35552-HPHNM

5 Fr. 2 Lumen 55 cm catheter length .018 inch dia. spring-wire guide

# Pressure Injectable Two-Lumen PICC with 80 cm Wire

**Contents:**

- 1: Two Lumen TaperFree™ Catheter†††: 5 Fr. (1.81 mm OD) x 55 cm Pressure Injectable, Contamination Guard
- 1: GlideThru™ Peel-Away Sheath: 5 Fr. x 4" (10 cm) Radiopaque over 5 Fr. Dilator
- 1: Spring-Wire Guide, Nitinol, Marked: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on One End - Straight Stiff Tip on Other)
- 1: Guidewire, Marked: .018" (0.46 mm) dia. x 80 cm Hydrophilic-Coated Nitinol with Soft Tip Tungsten Coil (CE 0120)
- 1: Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- 1: Syringe: 10 mL Luer-Lock
- 2: Dust Cap: Non-Vented
- 1: Catheter Trimmer
- 1: Safety Scalpel: #11
- 1: Patient ID Card
- 1: Patient Information Booklet
- 1: Paper Tape Measure
- 1: Dressing: STATLOCK®<sup>1</sup> PICC Plus Catheter Stabilization Device

<sup>1</sup>A registered trademark of C. R. Bard, Inc.

All components are CE 2797 unless otherwise noted.

**Rx only**

**Warning:** Read all package insert warnings, precautions, and instructions prior to use. Failure to do so may result in severe patient injury or death. [www.teleflex.com/IFU](http://www.teleflex.com/IFU)

California Prop. 65

**⚠ WARNING:** Cancer and Reproductive Harm.

[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

###MR Safe

Not made with natural rubber latex.

Fluid path components are non-pyrogenic.

**Contraindications:** The Pressure Injectable PICC is contraindicated wherever there is presence of device related infections or presence of thrombosis in the intended insertion vessel or catheter pathway. Clinical assessment of the patient must be completed to ensure no contraindications exist.



Lumen

Lumen	Priming Volume* (mL)	Gravity Flow Rate† (mL/hr)	Pump Flow Rate†† (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Pink (18 Ga.)	0.55	390	3420	5
White (18 Ga.)	0.55	390	3400	5

\* Priming volumes are approximate and are done without accessories.

† Flow rate values are approximate and are determined using deionized water at 100 cm head height.

†† Pump flow rates are determined at maximum pump pressure of 10 psig and represent approximate flow capabilities.

\*\* Pressure injection flow rates are determined at the injector pressure setting of 300 psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing.



CAS # 7440-48-4

###MR

EU Authorized Representative and Importer:

EC REP Teleflex Medical  
IDA Business and  
Technology Park  
Dublin Road, Athlone  
Co. Westmeath, Ireland

Arrow International LLC  
Subsidiary of Teleflex Incorporated  
3015 Carrington Mill Blvd.  
Morrisville, NC 27560 USA

Teleflex®  
[www.teleflex.com](http://www.teleflex.com)